



- IT** → Settaggi appropriati per gli inserti
- EN** → Appropriate settings for the inserts
- DE** → Korrekte Einstellungen für die Instrumente
- FR** → Réglages appropriés pour les inserts
- ES** → Ajustes apropiados para los insertos
- SV** → Korrekta Inställningar för spetsarna

INSERT	FUNCTION
--------	----------

IMPLANT PREP

IM1S	special*
IM1 AL	implant
IM1 SP	implant
IM2A	implant
IM2A-15	implant
IM2P	implant
IM2P-15	implant
IM3A	implant
IM3A -15	implant
IM3P	implant
IM3P-15	implant
IM4A	implant
IM4P	implant
P2-3	implant
P3-4	implant

MINI IMPLANT PREP

MDI 1.9	implant
MDI 2.2	implant
MDI 2.5	implant

OSTEOTOMY

OT1	cortical / cancellous
OT1A	cortical / cancellous
OT2	cortical / cancellous
OT3	cortical / cancellous
OT4	cortical / cancellous
OT5	cortical / cancellous
OT5A	cortical / cancellous
OT5B	cortical / cancellous
OT6	cortical / cancellous
OT7	cortical / cancellous
OT7A	cortical / cancellous
OT7S-3	special*
OT7S-4	special*
OT7-20	cortical / cancellous
OT8L	cortical / cancellous
OT8R	cortical / cancellous
OT9	cortical / cancellous

INSERT	FUNCTION
--------	----------

OSTEOPLASTY

OP1	cortical / cancellous
OP2	cortical / cancellous
OP3	cortical / cancellous
OP3A	cortical / cancellous
OP4	cortical / cancellous
OP5	perio
OP6	perio
OP6A	perio
OP7	cortical / cancellous

SINUS MEMBRANE ELEVATION

EL1	endo*
EL2	endo*
EL3	endo*

ENDO RETRO

EN1	endo*
EN2	endo*
EN3	endo*
EN4	endo*
EN5R	endo*
EN5L	endo*
EN6R	endo*
EN6L	endo*

EXTRACTION

EX1	cortical / cancellous
EX2	cortical / cancellous
EX3	cortical / cancellous

PERIO

PP1	perio
PS1	perio
PS2	perio
PS6	perio
PP10	perio
PP11	perio
PP12	perio

IMPLANT CLEANING

ICP + IC1	endo / perio
-----------	--------------

RESTORATIVE

DB2	special perio
-----	------------------

* Massima potenza consentita
* Maximum power permitted

* Maximal zugelassene Leistung
* Puissance maximum autorisée

* Máxima potencia permitida
* Maximalt tillåten effekt



- IT** → Settaggi appropriati per gli inserti
- EN** → Appropriate settings for the inserts
- DE** → Korrekte Einstellungen für die Instrumente
- FR** → Réglages appropriés pour les inserts
- ES** → Ajustes apropiados para los insertos
- SV** → Korrekta Inställningar för spetsarna

INSERT	MODE	POWER
--------	------	-------

IMPLANT PREP

IM1S	BONE	SPECIAL*
IM1 AL	IMPL	IMPLANT
IM1 SP	IMPL	IMPLANT
IM2A	IMPL	IMPLANT
IM2A-15	IMPL	IMPLANT
IM2P	IMPL	IMPLANT
IM2P-15	IMPL	IMPLANT
IM3A	IMPL	IMPLANT
IM3A -15	IMPL	IMPLANT
IM3P	IMPL	IMPLANT
IM3P-15	IMPL	IMPLANT
IM4A	IMPL	IMPLANT
IM4P	IMPL	IMPLANT
P2-3	IMPL	IMPLANT
P3-4	IMPL	IMPLANT

MINI IMPLANT PREP

MDI 1.9	IMPL	IMPLANT
MDI 2.2	IMPL	IMPLANT
MDI 2.5	IMPL	IMPLANT

OSTEOTOMY

OT1	BONE	CORTICAL / SPONGIOUS
OT1A	BONE	CORTICAL / SPONGIOUS
OT2	BONE	CORTICAL / SPONGIOUS
OT3	BONE	CORTICAL / SPONGIOUS
OT4	BONE	CORTICAL / SPONGIOUS
OT5	BONE	CORTICAL / SPONGIOUS
OT5A	BONE	CORTICAL / SPONGIOUS
OT5B	BONE	CORTICAL / SPONGIOUS
OT6	BONE	CORTICAL / SPONGIOUS
OT7	BONE	CORTICAL / SPONGIOUS
OT7A	BONE	CORTICAL / SPONGIOUS
OT7S-3	BONE	SPECIAL*
OT7S-4	BONE	SPECIAL*
OT7-20	BONE	CORTICAL / SPONGIOUS
OT8L	BONE	CORTICAL / SPONGIOUS
OT8R	BONE	CORTICAL / SPONGIOUS
OT9	BONE	CORTICAL / SPONGIOUS

INSERT	MODE	POWER
--------	------	-------

OSTEOPLASTY

OP1	BONE	CORTICAL / SPONGIOUS
OP2	BONE	CORTICAL / SPONGIOUS
OP3	BONE	CORTICAL / SPONGIOUS
OP3A	BONE	CORTICAL / SPONGIOUS
OP4	BONE	CORTICAL / SPONGIOUS
OP5	ROOT	PERIO
OP6	ROOT	PERIO
OP6A	ROOT	PERIO
OP7	BONE	CORTICAL / SPONGIOUS

SINUS MEMBRANE ELEVATION

EL1	ROOT	ENDO*
EL2	ROOT	ENDO*
EL3	ROOT	ENDO*

ENDO RETRO

EN1	ROOT	ENDO*
EN2	ROOT	ENDO*
EN3	ROOT	ENDO*
EN4	ROOT	ENDO*
EN5R	ROOT	ENDO*
EN5L	ROOT	ENDO*
EN6R	ROOT	ENDO*
EN6L	ROOT	ENDO*

EXTRACTION

EX1	BONE	CORTICAL / SPONGIOUS
EX2	BONE	CORTICAL / SPONGIOUS
EX3	BONE	CORTICAL / SPONGIOUS

PERIO

PP1	ROOT	PERIO
PS1	ROOT	PERIO
PS2	ROOT	PERIO
PS6	ROOT	PERIO
PP10	ROOT	PERIO
PP11	ROOT	PERIO
PP12	ROOT	PERIO

IMPLANT CLEANING

ICP + IC1	ROOT	ENDO / PERIO
-----------	------	--------------

RESTORATIVE

DB2	BONE	SPECIAL
	ROOT	PERIO

* Massima potenza consentita
* Maximum power permitted

* Maximal zugelassene Leistung
* Puissance maximum autorisée

* Máxima potencia permitida
* Maximalt tillåten effekt



- IT** → Settaggi appropriati per gli inserti
- EN** → Appropriate settings for the inserts
- DE** → Korrekte Einstellungen für die Instrumente
- FR** → Réglages appropriés pour les inserts
- ES** → Ajustes apropiados para los insertos
- SV** → Korrekta Inställningar för spetsarna

INSERT	MODE	POWER**
IMPLANT PREP		
IM1S	BONE	SPECIAL*
IM1 AL	BONE	1 / 2 / 3
IM1 SP	BONE	1 / 2 / 3
IM2A	BONE	1 / 2 / 3
IM2A-15	BONE	1 / 2 / 3
IM2P	BONE	1 / 2 / 3
IM2P-15	BONE	1 / 2 / 3
IM3A	BONE	1 / 2 / 3
IM3A -15	BONE	1 / 2 / 3
IM3P	BONE	1 / 2 / 3
IM3P-15	BONE	1 / 2 / 3
IM4A	BONE	1 / 2 / 3
IM4P	BONE	1 / 2 / 3
P2-3	BONE	1 / 2 / 3
P3-4	BONE	1 / 2 / 3

INSERT	MODE	POWER**
MINI IMPLANT PREP		
MDI 1.9	BONE	1 / 2 / 3
MDI 2.2	BONE	1 / 2 / 3
MDI 2.5	BONE	1 / 2 / 3

INSERT	MODE	POWER**
OSTEOTOMY		
OT1	BONE	1 / 2 / 3
OT1A	BONE	1 / 2 / 3
OT2	BONE	1 / 2 / 3
OT3	BONE	1 / 2 / 3
OT4	BONE	1 / 2 / 3
OT5	BONE	1 / 2 / 3
OT5A	BONE	1 / 2 / 3
OT5B	BONE	1 / 2 / 3
OT6	BONE	1 / 2 / 3
OT7	BONE	1 / 2 / 3
OT7A	BONE	1 / 2 / 3
OT7S-3	BONE	SPECIAL*
OT7S-4	BONE	SPECIAL*
OT7-20	BONE	1 / 2 / 3
OT8L	BONE	1 / 2 / 3
OT8R	BONE	1 / 2 / 3
OT9	BONE	1 / 2 / 3

INSERT	MODE	POWER**
OSTEOPLASTY		
OP1	BONE	1 / 2 / 3
OP2	BONE	1 / 2 / 3
OP3	BONE	1 / 2 / 3
OP3A	BONE	1 / 2 / 3
OP4	BONE	1 / 2 / 3
OP5	ROOT	PERIO
OP6	ROOT	PERIO
OP6A	ROOT	PERIO
OP7	BONE	1 / 2 / 3

INSERT	MODE	POWER**
SINUS MEMBRANE ELEVATION		
EL1	ROOT	ENDO*
EL2	ROOT	ENDO*
EL3	ROOT	ENDO*

INSERT	MODE	POWER**
ENDO RETRO		
EN1	ROOT	ENDO*
EN2	ROOT	ENDO*
EN3	ROOT	ENDO*
EN4	ROOT	ENDO*
EN5R	ROOT	ENDO*
EN5L	ROOT	ENDO*
EN6R	ROOT	ENDO*
EN6L	ROOT	ENDO*

INSERT	MODE	POWER**
EXTRACTION		
EX1	BONE	1 / 2 / 3
EX2	BONE	1 / 2 / 3
EX3	BONE	1 / 2 / 3

INSERT	MODE	POWER**
PERIO		
PP1	ROOT	PERIO
PS1	ROOT	PERIO
PS2	ROOT	PERIO
PS6	ROOT	PERIO
PP10	ROOT	PERIO
PP11	ROOT	PERIO
PP12	ROOT	PERIO

INSERT	MODE	POWER**
IMPLANT CLEANING		
ICP + IC1	ROOT	ENDO / PERIO

INSERT	MODE	POWER**
RESTORATIVE		
DB2	BONE	SPECIAL
	ROOT	PERIO

* Massima potenza consentita
* Maximum power permitted

* Maximal zugelassene Leistung
* Puissance maximum autorisée

* Máxima potencia permitida
* Maximalt tillåten effekt

** POWER: 1 = max 2 = med 3 = min



- IT** → Settaggi appropriati per gli inserti
- EN** → Appropriate settings for the inserts
- DE** → Korrekte Einstellungen für die Instrumente
- FR** → Réglages appropriés pour les inserts
- ES** → Ajustes apropiados para los insertos
- SV** → Korrekta Inställningar för spetsarna

INSERT	MODE	POWER
IMPLANT PREP		
IM1S	BOOSTED	C
IM1 AL	BOOSTED	C
IM1 SP	BOOSTED	C
IM2A	BOOSTED	C
IM2A-15	BOOSTED	C
IM2P	BOOSTED	C
IM2P-15	BOOSTED	C
IM3A	BOOSTED	C
IM3A -15	BOOSTED	C
IM3P	BOOSTED	C
IM3P-15	BOOSTED	C
IM4A	BOOSTED	C
IM4P	BOOSTED	C
P2-3	BOOSTED	C
P3-4	BOOSTED	C
MINI IMPLANT PREP		
MDI 1.9	BOOSTED	C
MDI 2.2	BOOSTED	C
MDI 2.5	BOOSTED	C
OSTEOTOMY		
OT1	BOOSTED	C
OT1A	BOOSTED	C
OT2	BOOSTED	C
OT3	BOOSTED	C
OT4	BOOSTED	C
OT5	BOOSTED	C
OT5A	BOOSTED	C
OT5B	BOOSTED	C
OT6	BOOSTED	C
OT7	BOOSTED	C
OT7A	BOOSTED	C
OT7S-3	BOOSTED	C
OT7S-4	BOOSTED	C
OT7-20	BOOSTED	C
OT8L	BOOSTED	C
OT8R	BOOSTED	C
OT9	BOOSTED	C

INSERT	MODE	POWER
OSTEOPLASTY		
OP1	BOOSTED	C
OP2	BOOSTED	C
OP3	BOOSTED	C
OP3A	BOOSTED	C
OP4	BOOSTED	C
OP5	HIGH	2 - 3
OP6	HIGH	2 - 3
OP6A	HIGH	2 - 3
OP7	BOOSTED	C
SINUS MEMBRANE ELEVATION		
EL1	LOW	1
EL2	LOW	1
EL3	LOW	1
ENDO RETRO		
EN1	LOW	1
EN2	LOW	1
EN3	LOW	1
EN4	LOW	1
EN5R	LOW	1
EN5L	LOW	1
EN6R	LOW	1
EN6L	LOW	1
EXTRACTION		
EX1	BOOSTED	C
EX2	BOOSTED	C
EX3	BOOSTED	C
PERIO		
PP1	HIGH	2 - 3
PS1	HIGH	2 - 3
PS2	HIGH	2 - 3
PS6	HIGH	2 - 3
PP10	HIGH	2 - 3
PP1C	HIGH	2 - 3
PP12	HIGH	2 - 3
IMPLANT CLEANING		
ICP + IC1	HIGH	2 - 3
	LOW	1
RESTORATIVE		
DB2	BOOSTED	A
	HIGH	2 - 3

* Massima potenza consentita
* Maximum power permitted

* Maximal zugelassene Leistung
* Puissance maximum autorisée

* Máxima potencia permitida
* Maximalt tillåten effekt