

- SETTAGGI APPROPRIATI PER GLI INSERTI
- APPROPRIATE SETTINGS FOR THE INSERTS
- KORREKTE EINSTELLUNGEN FÜR DIE INSTRUMENTE
- RÉGLAGES APPROPRIÉS POUR LES INSERTS
- AJUSTES APROPIADOS PARA LOS INSERTOS
- KORREKTA INSTÄLLNINGAR FÖR SPETSARNA

→ COMBI touch



function	endo	perio/scaler	restorative
SCALING			
S1, S2, S3, S4, S5, S6, S7, S8		3-5	
PERIO			
P1, P2, P3, P4, P10, P11, P12, P13, P14		1-3	
IMPLANT CLEANING			
ICS		1	
ENDO			
E1, E2	1-5		
ENDO REVISION			
ER1		1-5	
ER2, ER3, ER4, ER5		1-3	
ENDO RETRO			
R1, R2, R3, R4, R5, RD3, RD4, RD5		1-3	
EXTRACTION			
ME1, ME2, ME3			5
CROWN PREP			
DB1			3-5
CAVITY MARGIN			
CM1, CM2, CM3, CM4			1-4
CAVITY PREP (available only in Europe)			
CP1, CP2, CP3, CP4			1-3
CP5, CP6			1-4
RESTORATIVE			
D1, D2, D4			1-5
D3			1-3

