

# (EN) English: Instructions for Use

## Product description:

**GINGISAGE regular** is a gingival haemostatic retraction paste.

When inserted in the sulcus it leads to temporary retraction of the marginal gingiva.

As a result of the pressure exerted on the tissues and the presence of ferric chloride (astringent substance) GINGISAGE regular makes it possible to have very good control of bleeding and gingival fluids.

May be used in combination with a retraction cord and after any gingival evulsion procedure

This paste may be applied with a commercial capsule dispenser gun.

GINGISAGE regular is contained in a capsule for use on a single patient: capsules must not be re-used.

## Indications:

All procedures requiring the sulcus to be opened and/or dried.

Joint prosthesis impression by standard or digital technique.

Cementing of prostheses.

Filling of cavities by composites or other materials.

## Contraindications:

Do not use in the case of poor periodontal condition: furcations, deep pockets.

## Precautions:

The practitioner must wear protective glasses.

If there is contact with the eyes, rinse copiously with water and consult a specialist if irritation persists.

## Use:

Tear the packaging, remove the capsule and insert it into the dispenser gun. Fig. 1, 2, 3

Remove the sealing cap from the capsule. Fig. 4

Remove the first mm of paste extruded. Fig. 5

Blow away the excess blood or saliva with the air spray.

Inject the paste into the sulcus using the tooth as support and fill the sulcus. Fig. 6, 7

Inject slowly controlling the bleaching of the marginal gingiva. Fig 8

Leave in place for 1 to 2 min away from any saliva. Fig. 9

Remove the paste using an air and water spray and a suction device simultaneously. Fig. 10

All the paste must be removed and must not be swallowed by the patient.

After use the empty capsule should be discarded with general waste. The dispenser gun should be cleaned and disinfected.

**Storage:**

15°C to 25°C 59 to 77°F

Do not use beyond the expiry date.

**Guarantee:**

KEREDEN guarantees the product against any manufacturing defect or fault. The user is responsible for its use.

**Limitation of liability:**

The guarantee provided by KEREDEN is limited to replacement of the product found to be defective with the exclusion of any penalty or compensation.

KEREDEN cannot be held liable for any damage whether direct or indirect, material or immaterial, consequential or not.

The guarantee is without prejudice to the provisions relating to the legal guarantee.

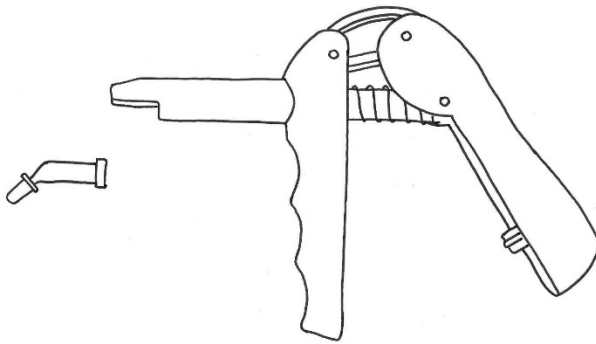


Abbildung 21

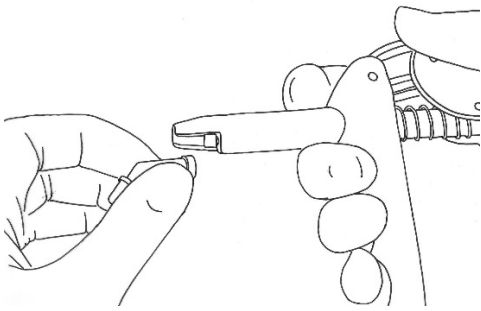


Abbildung 22

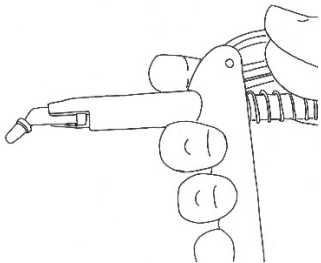


Abbildung 23

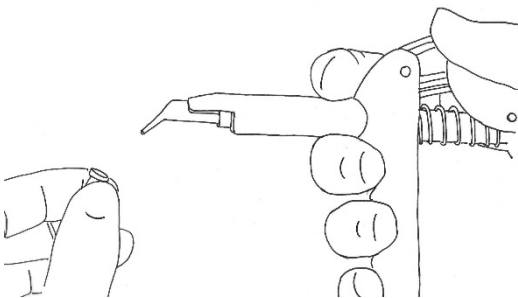


Abbildung 24

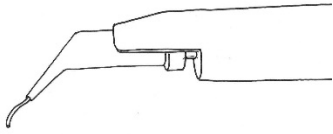


Abbildung 25

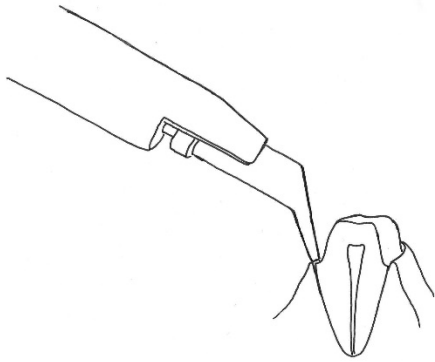


Abbildung 26

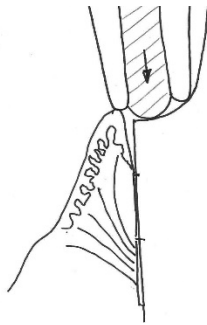


Abbildung 27