

MECTRON PROPHYLAXIS P3 - THE NEW FULL SCOPE MAINTENANCE INSERT




new!

mectron

NEW PERIO INSERT P3

Specially designed to gently and effectively remove supra and subgingival biofilm.

- Its slim shape allows an easy access in deep periodontal pockets and interproximal spaces in all the quadrants.
- Thanks to its remarkably linear movement it is minimally invasive for the soft tissues and root cementum and preserves the epithelium.
- The best interproximal and subgingival cleaning.
- It effectively removes extrinsic stains.
- It gives the clinician an optimal intraoperative control and a secure feeling while working interproximally, supra and subgingivally.
- Pleasant and comfortable treatment for the patient thanks to the SOFT MODE function.

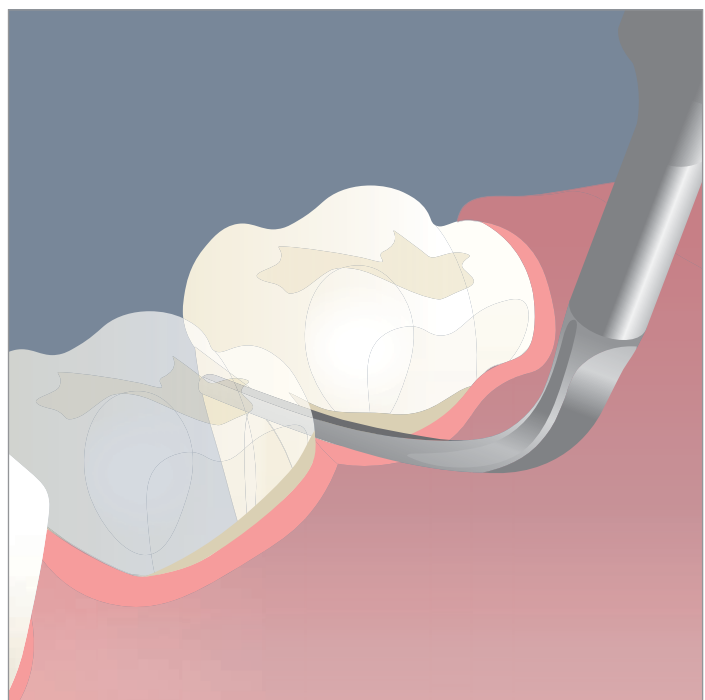


→ CORRECT USAGE OF THE P3 INSERT

The linear insert motion allows to preserve the soft tissues and the gingival margin.

The force applied by the tip to the tooth should be feather-light.

It's recommended to set the power to a low level with medium irrigation. If available, activate the SOFT MODE function, which reduces the amplitude of the ultrasound oscillations and allows the insert to move gently and efficiently.

More sensitive patients will benefit greatly from this mode and will be more likely to trust their dentists.

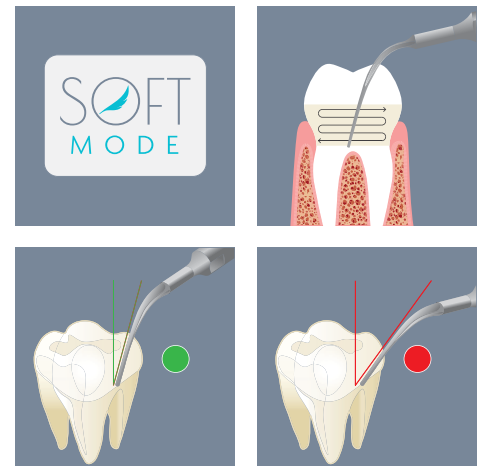

by courtesy of Dr Giacomo Oldoini (La Spezia, Italy)

Our KOLs confirm its effectiveness, safety and comfort in various situations.

“Its slim and long shape allows an easy access in areas difficult to reach. The best performance in the biofilm removal both in supra gingival areas and periodontal pockets always guarantees the maximum respect for the soft tissues. Particularly suitable for the treatment of orthodontic patients; it works perfectly around the brackets avoiding any risk of damage.”

Dr Stefano Checchi, Torino, Italy

“Thanks to its features it is particularly suitable for the initial preparation and the periodontal maintenance. Long and curved, it allows an easy access in periodontal pockets with different depths, also in case of thin gingival phenotypes. Particularly suitable for the subgingival biofilm management.”

Dr Giacomo Oldoini, La Spezia, Italy

→ P3 INSERT IN DETAIL



P3 INSERT Ref. n° 02970003-001

slim and smooth like a probe

Indications:

supra and subgingival concretions and
biofilm removal from interproximal spaces

SAFEKEY K10

Every insert is supplied with its own torque wrench to avoid cross contamination and to reduce the sterilization time.



mectron s.p.a.
via Loreto 15/A, 16042 Carasco (Ge), Italy, tel +39 0185 35361,
fax +39 0185 351374, www.mectron.com, mectron@mectron.com



© Copyright mectron S.p.A., Carasco, Italy
All rights reserved. Texts, pictures and graphics of mectron brochures are protected by copyright and other protection laws. Without written approval of mectron S.p.A. the contents may not be copied, distributed, changed or made available to third parties for commercial purposes.

mectron
medical technology