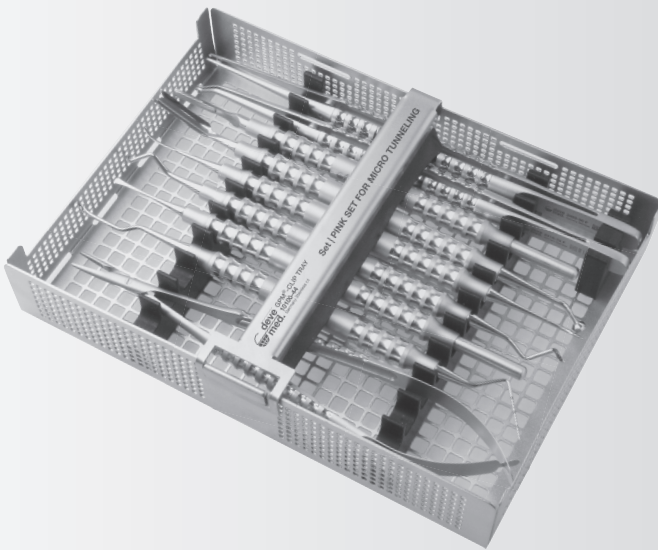
... makes the difference

**RECOMMENDATION OF
DR. ANA LUISA BERNOTTI**

DR. ANA LUISA BERNOTTI

DDS. Periodontist – Implantologist.

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- Periodontist 1999. Universidad Central de Venezuela
- Implantologist 2015. Centro de especialidades y Estudios
- Implantologist 2015 . CEYESOV . Mexico.
- Active Member of Periodontology Society of Venezuela. Since 2000.
- Active Member of Iberoamerican Federation of Periodontology. Since 2000.
- Author of "V-Y Bernotti Flap". Bernotti- Crocetti 2016.
- National and International Speaker.
- Private Practice: Bernotti Odontologia Avanzada". Caracas - Venezuela.
- Active member "Follow The Pink" & "Divas in Dentistry" website

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PPS | Soft Tissue Extender / Periodontal Probe

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The periosteum can be loosened and expanded by short, pulling movements exerted with slight pressure. This technique ensures an optimal blood supply, reduces swelling as well as pain and minimizes injuries to the vessels. With the opposite periodontometer, one can dispense with a separate depth gauge.

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The workpiece ends are arrow-head shaped and angulated specifically to allow for lateral tunnel procedures in difficult to access approximal regions. The papillae are gently tunneled by using the very flat workpieces. To avoid injury to the gingiva, the side of the working part which faces the gum is blunt.

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The lance-shaped end of the workpiece enables the detachment of the papilla as well as of the marginal gingiva crest. With the slightly curved working part the gingiva is suprapariostally removed/loosened - the rounding of the working part prevents the mucosal flap from being pierced.

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This instrument combines the plugger, applicator and retractor in one. The spatula-shaped working part can be used to attach bone material etc. and at the same time to retract the gingiva.

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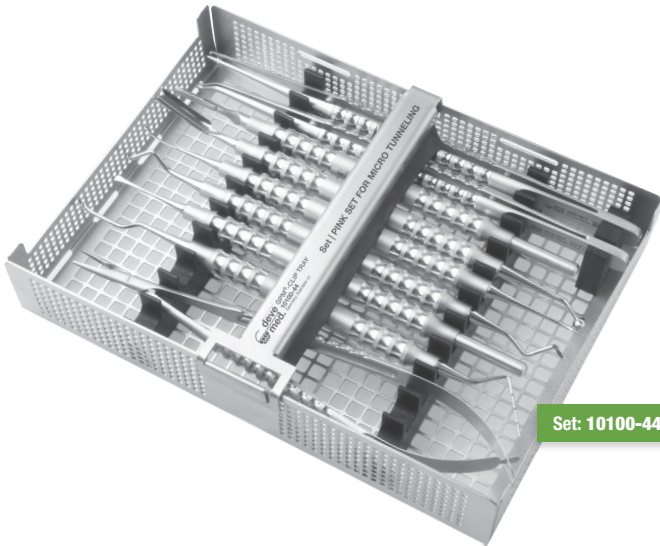
Bernotti 17,1 cm

PINK SET FOR MICRO TUNNELING nach Dr. Ana Luisa Bernotti

PINK SET FOR MICRO TUNNELING in accordance to Dr. Ana Luisa Bernotti

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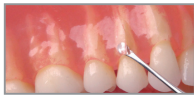
The gingivectomy knife can be used as a micro-blade for the tunnel technique due to its flexible, sharp working end.

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The workpiece ends are arrow-head shaped and angulated specifically to allow for lateral tunnel procedures in difficult to access approximal regions. The papillae are gently tunnelled by using the very flat workpieces. To avoid injury to the gingiva, the side of the working part which faces the gum is blunt.

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PPS 1700 F **F-line**
Bernotti | 17,1 cm

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Scalpel blade holders | single-ended

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devemed GmbH